**AT03107**

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EN

Thank you for purchasing our product. Please read the user instructions carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare professional or the seller to avoid damaging the product or causing injury.

**WARNING:** You must check all parts for damage that may have occurred during transportation. If any damage is noticed, DO NOT use the product. For further information, please contact the seller.

#### **TARGET PATIENT GROUP**

Individuals dealing with illnesses, dysfunctions, or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the intended use of the product in this user manual). The product can be purchased by the user independently or based on the recommendations of a doctor, therapist, or other specialist. Whether the product is purchased independently or recommended by a doctor/therapist/other specialist, you must consider the available sizes/necessary functions and variants of the product, as well as the indications and contraindications for use, and the information provided by the manufacturer.

#### **USE/INDICATIONS**

The three-part rehabilitation mattress is a medical device used in physiotherapy, rehabilitation, and corrective exercises. Its design allows the use of various therapeutic techniques aimed at supporting proper posture and minimizing the risk of injury. It can be used both in clinics and for home exercises, as well as as an aid in motor skills development for children.

Indications:

Post-injury and post-surgical rehabilitation – The mattress provides excellent support during the recovery process from muscle, joint, and bone injuries, as well as after surgical procedures, especially orthopedic ones.

Manual therapy and physiotherapy – Used in rehabilitation clinics as a base for manual therapy, massage, and posture-correcting exercises.

Corrective exercises for children and adults – Helps in corrective gymnastics, supports therapy for postural defects (e.g., scoliosis, lordosis, kyphosis), and strengthens the muscles stabilizing the spine.

Therapy for individuals with limited mobility – Enables safe exercise for elderly people, patients recovering from strokes, or those with neurological issues, improving their range of motion and coordination.

Prevention and therapy for back pain – Provides support for stretching and strengthening exercises for the back muscles, helping to reduce back pain, especially caused by a sedentary lifestyle.

General fitness and strengthening exercises – Can be used for various physical activities such as Pilates, stretching, yoga, stabilizing exercises, and deep muscle strengthening.

Sensory therapy and motor development for children – Helps with the development of gross motor skills in children, supporting their physical development through various exercises and movement games.

Support for relaxation and breathing exercises – Facilitates the performance of relaxation techniques, breathing exercises, and stretching, which is particularly helpful in reducing stress and muscle tension.

Use in home rehabilitation – Ideal for individuals who need a comfortable surface for exercises at home without the need to visit professional clinics.

Injury prevention during training – Provides cushioning and safety during exercises, reducing the risk of injuries from falls or incorrect movements.

#### **CONTRAINDICATIONS**

Physical or mental limitations (e.g., visual impairment) that prevent the safe use of the product.

Acute injuries and inflammation – Using the mattress may not be recommended for individuals with acute joint, muscle, or soft tissue inflammation, or for those with fresh injuries, fractures, or sprains.

Advanced osteoporosis – People with advanced osteoporosis should consult a doctor before using the mattress to avoid the risk of fractures or strain on the spine.

Neurological diseases with balance disorders – Patients with neurological diseases (e.g., ataxia, Parkinson's disease, epilepsy) may struggle to control their body during exercises on the mattress, increasing the risk of falls and injuries.

Severe mobility restrictions and lack of body control – Individuals who are completely immobilized or have severe limb paralysis may require specialized equipment that provides adequate stabilization.

Advanced cardiovascular and respiratory diseases – Physical activity on the mattress may be dangerous for patients with heart failure, advanced hypertension, or lung diseases, so medical consultation is necessary in such cases.

Wounds, bedsores, and skin changes – Using the mattress may be painful or worsen skin conditions in the presence of open wounds, bedsores, or skin irritations.

Post-surgical recovery period – Individuals who have had surgery, especially orthopedic or spinal surgery, should avoid intensive exercises on the mattress without prior medical consent.

#### **LIST OF COMPONENTS**

Three-part rehabilitation mattress, user manual.

#### **PRODUCT FEATURES**

Polyurethane foam with dimensions 85 x 65 x 5cm

Waterproof laminated cover

Weight: 3,15 kg

#### **USAGE**

Preparation for use

Place the mattress on a stable, flat surface.

Ensure there are no objects around that could restrict movement or pose a danger.

Before use, check that the mattress is free of damage, abrasions, or deformities.

Usage

Rehabilitation and therapeutic exercises

Use the mattress for exercises recommended by a physiotherapist, tailored to your health needs.

You can perform strengthening, stretching, stabilizing exercises, as well as joint mobility exercises.

Manual therapy and massage

The mattress provides a comfortable base for manual therapy and massage, ensuring comfort for both the patient and therapist.

It can be used as a place for techniques that relax muscles and improve tissue flexibility.

Corrective gymnastics and postural exercises

Used in the prevention and therapy of postural defects, it supports proper spinal and limb alignment.

It can be used for exercises that improve balance and motor coordination.

Physical activity for children

The mattress can be used as a safe space for physical activities that support the development of gross motor skills.

Children can practice rolls, jumps, or climb on the folded sections of the mattress.

Relaxation and stretching exercises

You can use the mattress for yoga, Pilates, and breathing exercises, which help reduce stress and muscle tension.

#### **PRECAUTIONS**

Avoid using the mattress on a slippery surface – if there is a risk of the mattress sliding, place an anti-slip mat underneath it.

Do not use the mattress near open flames or hot objects – this could damage its structure.

Do not stand on the edge of the mattress in a vertical position – this could cause it to tip over or get damaged.

Children should use the mattress under adult supervision – especially during intense physical play.

**WARNING:**

In the event of a related “serious incident” that directly or indirectly led, could have led, or may lead to any of the following:

- a) death of the patient, user, or any other person, or
- b) temporary or permanent deterioration of the patient’s, user’s, or any other person’s health, or
- c) serious threat to public health,

the above “serious incident” should be reported to the manufacturer and the competent authority of the member state where the user or patient resides. In Poland, the relevant authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

**WARNING:**

If you experience pain, allergic reactions, or other troubling or unclear symptoms related to the use of the medical device, consult a healthcare worker.

**WARNING:**

The product must not be used in any manner other than its intended use.

#### **CLEANING**

Cleaning the Cover:

Can be machine washed at a maximum temperature of 95°C

Do not iron

Do not dry clean

Do not tumble dry

Do not bleach

Cleaning the Foam:

**WARNING:** Do not machine wash the foam!

Cleaning the foam requires a delicate approach to avoid damaging its structure. Here's a guide on how to clean it properly:

What you will need:

Mild detergent or dishwashing liquid

Warm water

Baking soda (optional for deodorizing)

Vacuum cleaner (preferably with a soft brush)

Spray bottle

Soft cloth or sponge

Enzyme cleaner (optional for stain removal)

Cleaning Steps:

Vacuum the foam:

Remove dust and debris: Use a vacuum cleaner with a soft brush to gently vacuum the foam surface. This will remove dust, dirt, and debris before wet cleaning.

Spot cleaning with mild detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Gently spray the foam: Avoid soaking the foam. Lightly spray the mixture on the soiled area.

Dry the stains: Use a soft cloth or sponge to gently blot the area. Do not rub, as it may damage the foam structure.

Rinse the area: If necessary, lightly spray with clean water to remove detergent residues.

Dealing with tough stains:

Biological or urine stains: Use an enzyme cleaner. Apply the cleaner to the stain, leave it for a moment (according to the product instructions), and then blot it with a clean cloth.

Greasy stains: Use a small amount of dishwashing liquid or a special upholstery cleaner designed for foam materials.

Deodorizing (optional):

Use baking soda: Sprinkle a layer of baking soda on the foam and leave it for several hours or overnight to absorb odors.

Vacuum the baking soda after sufficient time has passed.

Drying the Foam:

Air dry thoroughly: Foam is highly absorbent, so it may take some time to dry. Leave it in a well-ventilated area.

Use a fan to speed up the process: Never use a direct heat source, such as a hair dryer, as it may damage the foam. It is best to use a cool fan.

Make sure the foam is completely dry before reuse to prevent mold or fungus growth.

What to avoid:

Never immerse the foam in water, as it may be difficult to dry and could be damaged.

Avoid using harsh chemicals, such as bleach or alcohol-based cleaning agents, which could weaken the foam structure.

No direct heat: Heat can damage the foam structure, so avoid drying in a dryer or using hot air.

#### **STORAGE**

Store the product in a dry place, away from direct sunlight.

**DISPOSAL**

Once the product is no longer in use, treat it as regular household waste. Remember to follow waste sorting regulations. Take care of the environment.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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