AT51026

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GENERAL WARNING

Thank you for purchasing our product. Please read the contents of this manual. Do not use the product until you have read and thoroughly understood the instructions. If you do not understand the warnings, cautions or instructions, please contact your healthcare professional or dealer to avoid damage to the product.

It is mandatory to read the contents of this manual before using the product. Do not use if the weight of the user exceeds 136 kg.

Store away from direct sunlight and heat source.

The toilet chair is designed for use on flat and level surfaces.

Standing on the toilet chair is not permitted.

Check the product regularly for damage and wear. If you notice any damage, stop using the product immediately and contact your dealer.

PRODUCT DESCRIPTION

The height-adjustable toilet chair is a modern product designed for people with limited mobility. Assembly, the product is quick, easy and tool-free. The product is made of a stable steel frame, equipped with soft comfortable armrests. The chair's legs are finished with non-slip feet to ensure stability. The set includes a bucket with lid and a toilet seat with lid. The product is easy to handle and convenient to fold for transport or storage.

PRODUCT CHARACTERISTICS:

- steel frame
- soft, comfortable armrests
- height adjustable
- anti-slip feet
- bucket with lid
- easy-to-clean toilet seat

APPLICATION

The toilet chair allows patients with limited mobility to attend to their physiological needs. It can be used in close proximity to the bed or patient area. In addition, the chair can be used without buckets over the toilet seat.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling .

NOTES:

In the event of a product-related 'serious incident' that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

WARNING: During use and handling of the product and when assembling and adjusting the mechanisms, there may be a danger of trapping and/or pinching parts of the user's/companion's body in the openings/gaps between components. Perform these operations with particular care. Once adjustments have been completed, stabilise the position by tightening the nuts/screws firmly.

NOTE: Before use, check that all components have been assembled correctly.

ATTENTION: it is forbidden to use the product other than for its intended purpose

NOTE: Before use, check that all components have been assembled correctly.

ATTENTION: The manufacturer is not responsible for damage caused by negligent maintenance or as a result of not following the instructions in this manual.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

USE

- 1 Remove the product from the packaging. Unfold the chair frame so that the side armrests are parallel to each other (Fig. 1.1).
2. To stabilise the chair frame, place the crossbar with the bucket space in the prepared latches. Check that the latches have properly locked into the frame lateral.
3. at the back of the chair, we place a railing in the appropriately prepared holes to prevent the toilet seat lid from falling backwards. The railing is mounted with snaps.
- 4 To change the height of the chair, adjust the height of each leg separately using the latch on each leg of the chair. Ensure that the legs are of identical height and stable before use.
- 5 Place the bucket in the space provided for it, between the metal railings.
- 6 To install the seat, press it against the rear transverse frame in two places. Then attach the toilet seat cover to the rear transverse frame.
- 7 Ensure that the plastic fixing clips are correctly attached and that the lid and toilet seat are in the correct position before use.

1.1

1.2

MAINTENANCE OF THE PRODUCT

1. empty and disinfect the container and toilet seat after each use to avoid contamination.
2. clean the chair frame with water and cleaning agents (avoid corrosive substances).
3. clean the armrests with mild detergents.

When not in use, the product should be stored in a dry room at room temperature, away from direct sunlight. It is recommended to store the product in its packaging.

TECHNICAL SPECIFICATIONS

Material: powder-coated steel, plastic

Total weight: 6.6 kg

Minimum height from ground to armrests: 58 cm

Overall width: 74 cm

Width between lower legs: 49 cm

Minimum seat height: 35 cm

Maximum seat height: 50 cm

Maximum user weight: 136 kg

**THIS SIGN INDICATES THE MAXIMUM
WEIGHT OF THE USER**

HOW TO DISPOSE OF THE PRODUCT

This medical device can be disposed of as normal municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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