AT51114

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Thank you for purchasing our product. Please read the contents of this manual. Do not use the product until you have read and thoroughly understood the instructions. If you do not understand the warnings, cautions or instructions, please contact your healthcare professional or dealer to avoid damage to the product.

GENERAL WARNING

- DO NOT assemble or use the product before reading and understanding these instructions.
- The product is not intended to be used in a sitting position.
- A doctor/therapist should be present when adjusting the height to ensure that the user is supported and that the screws are correctly adjusted and tightened. When the product is assembled, always ensure that the knobs for adjusting the handles are locked and the metal buttons are in the correct places. Failure to do so may lead to injury.
- Before use, check that all adjustments have been secured and that the castors and moving parts of the product are in good condition.
- When in use, the castors must be in contact with the ground at all times to ensure adequate stability.
- When the pulpit is used in a stationary position, the brakes must be in the locked position.
- Failure to do so may lead to tipping over and injury to the user.
- Always check that all components are securely locked in position before use.
- To maintain safety when using the pulpit, follow the guidelines below.
- Do not exceed the maximum load of the product, which is 136 kg.

NOTE: Inspect all parts for damage caused in transport. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

WARNING: During use and handling of the product and when assembling and adjusting the mechanisms, there may be a risk of entrapment and/or compression of parts of the user's/companion's body in openings/gaps between components. Perform these operations with particular care. Once the adjustment is complete, stabilise the position by tightening the nuts/bolts carefully.

CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of a patient, user or other person, or
 - (b) temporary or permanent deterioration of the health of a patient, user or other person, or
 - (c) a serious risk to public health
- the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In

the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

ATTENTION: It is forbidden to use the product other than in accordance with its intended use.

PURPOSE/USE

The product is intended for use by disabled people and people with reduced mobility. It supports and relieves the lower limbs during movement, in post-traumatic and post-operative conditions as well as during subsequent rehabilitation.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) which prevent safe handling of the product.

NOTE: The product is intended for indoor use.

TARGET PATIENT GROUP

People struggling with diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

DESCRIPTION

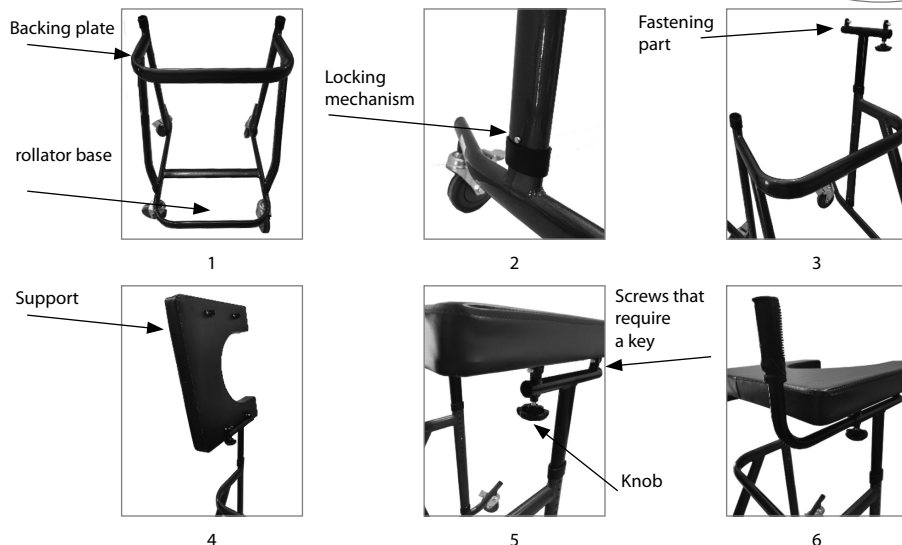
The frame is made of lightweight, surface-treated duralumin profiles, together with a padded backing plate and ergonomic plastic handles. The product is equipped with four swivel castors, the rear two of which are equipped with a foot brake. Easy height adjustment is made possible by the adjustable backing plate.

SCOPE OF DELIVERY

The product (consists of base, main frame, handles, support, knobs), instruction manual.

ASSEMBLING/PREPARING FOR USE:

1. Set the product base on the wheels, then apply the main frame (support plate), fitting all four connectors at the same time. Press down so that all four locking locks are pressed into place. (Figures 1 and 2)
- 2 Place the attachment part on the support plate in the corresponding hole (fix by pressing in the locking lock) and then the support. Tighten the support using the key provided. (Figures 3 and 4)

3 The support plate can be adjusted (11 degrees of adjustment) by placing the locking latches at the correct height.

4. Place the handles in the special holes under the support, loosen the knob and adjust to the correct length, tighten the knob underneath. (Figures 5 and 6)

5. To ensure the stability of the product in the rest position, foot brakes are used. The brakes are activated by pushing down the rear wheel pressure lever, which causes braking by locking the wheels.

DISASSEMBLING

Follow all steps described under unfolding/preparing for use in reverse order.

FOLDING AND ASSEMBLY WARNINGS

After assembling the product, make sure that both handles are at the same height and do not move.

TECHNICAL DATA

Total base width: 71 cm

Total base depth: 86 cm

Total height (from the handles to the ground): 117-145 cm

Support dimensions: Width: 62 cm, Depth: 49 cm, Height: 7 cm

Weight: 9.8 kg

Maximum load: 136 kg

This symbol indicates the maximum weight of the user!

MAINTENANCE AND CLEANING

The product can be cleaned and maintained with ordinary, solvent-free cleaners, disinfectants and preservatives. After cleaning, the product should be wiped dry. To maintain maximum stability, we recommend regularly checking the position of all fastening elements. The smooth rotation of the wheels should be checked regularly and debris removed if necessary.

WHEEL MAINTENANCE

1 Clean the castors with warm water and a mild detergent. Dry with a dry cloth.

2. Lubricate periodically with 3-in-1 grease if the castors begin to squeak.

DISPOSAL

Dispose of municipal waste in accordance with the applicable disposal conditions.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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