AT51124

**ROTOR REHABILITACYJNY INSTRUKCJA OBSŁUGI
REHABILITATION ROTOR USER MANUAL
REHABILITATION ROTOR BENUTZERHANDBUCH
NÁVOD K REHABILITAČNÍMU ROTOPEDU
NÁVOD NA POUŽITIE REHABILITAČNÝ ROTOPEĐ
MANUEL D'UTILISATION DU ROTOR DE RÉHABILITATION
GEBRUIKSAANWIJZING REVALIDATIEROTOR
MANUAL DEL USUARIO DEL ROTOR DE REHABILITACIÓN
MANUALE D'USO DEL ROTORE DI RIABILITAZIONE
BRUKSANVISNING FÖR REHABILITERINGSROTOR**

IU_AT51124

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GENERAL WARNINGS

Warning: Failure to observe the following safety instructions may result in injury or trauma that will not promote recovery.

Warning:

- 1) Do not stand on the pedals.
- 2) Sit on a chair, sofa or bed when exercising.
- 3) Ensure all screws, bolts, knobs, frames and brackets are fastened before using the product.
- 4) Patients, people with disabilities and the elderly must use it when accompanied by another person.
- 5) Persons suffering from heart disease or prohibited from strenuous exercise should follow their doctor's instructions and perform appropriate exercises.
- 6) This product should not be used by people with untreated injuries.
- 7) Children and people with disabilities should be closely supervised during the use of rotor.
- 8) This product is specifically designed for family use.
- 9) Special attention should be paid to safety issues to avoid injury when transporting the device.
- 10) Before using the product, perform appropriate stretching exercises to prevent injury during exercise.

NOTE: Inspect all parts for damage caused during transport. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

CAUTION:

In the event of a device-related „serious incident“ that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of the patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above „serious incident“ must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

CAUTION: During use and handling of the product, and when assembling and adjusting the mechanisms, there may be a danger of the user's/companion's body parts becoming trapped and/or compressed in the openings/gaps between

components. Perform these operations with particular care. Once adjustments have been completed, stabilise the position by tightening the nuts/bolts securely.

PURPOSE

The rehabilitation rotor is a compact exercise bike which can be easily stored when necessary. The rotor is used mainly for rehabilitation of people with limb paresis caused, among others, by an accident or stroke. When used regularly, it stimulates blood circulation and increases muscle strength. It is a great way to moderately exercise the cardiovascular system.

PRECAUTIONS DURING USE

Please read the following instructions carefully before using the product. For the following conditions, the user should take note of the exercise time and consult a doctor before using the product:

1. cardiovascular problems
2. sensory disorders
3. osteoporosis or diabetes
4. pregnancy
5. malignant tumour
6. high blood pressure and acute illnesses
7. patients with lumbar fractures; for example, those suffering from skeletal muscle diseases
8. arthritis
9. dizziness and tinnitus

DETAILED DESCRIPTION OF ACCESSORIES

- Rotor base x2
- U-brace x1
- T-column x1
- Straight column x1
- Element with pedals/grips
- Triangular knob x1
- Screw M8*40 mm x8
- Screw M8*60 mm x2
- Locking screw M8 x10
- Fork spanner x1
- L-wrench x1

Fig. 1

PARAMETERS

- Weight: 2.8 kg
- Rotor base: 40 cm x 40 cm
- Maximum height: 100 cm

Minimum height: 75 cm
 U-frame height: 20 cm
 Handle spacing: 42 cm
 Pedal spacing: 42 cm
 Maximum user weight: 120 kg

This symbol indicates the maximum weight of the user

Fig. 2

Fig. 3

Fig. 4

Fig. 5

Fig. 6

Fig. 7

USING

Step 1

Align the two base tubes (fig. 1) and then connect the U-frame to the rotor base, to do this use a M8*10 mm bolt with lock nut, paying attention to the direction of the upper sleeve. (fig. 2)

Step 2

Then attach the pedal component (including the foot bushing). Using a M8*40 mm screw. (fig. 3)

Step 3

Screw the triangular knob into the intended place, turn the lower knob to adjust the resistance during training. (fig. 3)

Step 4

Use M8*60 mm bolts and nuts to connect the straight column to the rotor base. Pay attention to the direction of the column and the concave connection. (fig. 4)

Step 5

Insert the column in a T-shape. Paying attention to the vertical direction of the T-shaped column, fix the T-shaped column with a clamp at the correct height. At the same time, use an L-wrench to tighten the adjustment screw on the side of the bracket. (Fig. 5)

Step 6

Use M*40 mm bolts and nuts to secure the top of the rotor, screw the triangular knob at the top. Turn the top knob to adjust the resistance during training. (fig. 6)

Step 7

Complete the assembly, make sure all screws are

properly tightened and adjust the height of the apparatus to suit your needs.

Precautions during use:

When using the product, try to keep your legs and arms relaxed. Wear comfortable clothing when exercising. Exercising in the opposite direction may cause injury to the legs and arms. Do not over-exert yourself. Place the apparatus on a flat floor. If the apparatus is standing on a wooden floor or carpet, place a mat underneath to avoid damage to the floor and carpet during exercise. Pay particular attention to safety issues to avoid injury when transporting the device.

CLEANING/MAINTENANCE

Clean with a damp cloth and soap.

In order to maintain the performance of the product, it is extremely important to keep all its components clean, as well as how it is stored. Storage in rooms with very high humidity (bathrooms, laundries, etc.), or outdoors, combined with poor hygiene, will result in a rapid loss of performance and aesthetics.

ATTENTION: Do not expose the product to extreme weather conditions (rain, snow very strong sun), as certain elements may change colour.

ATTENTION: The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or as a result of not following the recommendations contained in this user manual.

ATTENTION: It is forbidden to use the product other than in accordance with its intended use!

ATTENTION: All exercises should be performed in a sitting position, standing on the pedals is prohibited.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product. (if there are no specific, obvious ones, we give general ones)

DISPOSAL

Municipal waste, please dispose of in accordance with the applicable disposal conditions.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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