## AT52313

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Before utilizing your electric wheelchair for the first time, it is crucial to thoroughly read and adhere to all the information, warnings, and precautions outlined in this manual. Should any section be unclear or if you require further guidance regarding installation or operation, we strongly advise reaching out to your retailer or the manufacturer for assistance. The secure and proper usage of the product heavily relies on adhering to the information, precautions, and warnings provided in this manual. Please note that we cannot be held responsible for any damages or injuries incurred from the unsafe use of the product or the failure to follow the instructions, cautions, and warnings as outlined in this manual.

## 1. INTRODUCTION

Prioritize your safety by thoroughly reviewing and adhering to these usage instructions before using your power wheelchair for the first time. These instructions hold paramount importance. A comprehensive understanding of these guidelines is crucial for the safe and proper operation of the wheelchair.

### GENERAL WARNINGS

- Handle curbs carefully and make turns on hills.
- Avoid steep or uneven surfaces to prevent wheelchair tipping.
- Avoid very soft surfaces as the wheels may sink in.
- Move the wheelchair smoothly in all conditions.
- Keep both feet on footrests, avoid clothing blockage in wheels.
- Always drive carefully on inclines.
- Keep both hands on push handles for control (assistant only).
- Always remember to keep your balance.
- Regularly check wheelchair parts and follow manual instructions even post-warranty
- Keep the wheelchair away from sources of fire
- Under no circumstances should the wheelchair be overloaded.
- Avoid hanging heavy objects on the stroller's back to maintain stability, especially on hills
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Avoid lifting the front wheels while moving to prevent falls and serious injury
- Do not get into the wheelchair while it is moving.
- Do not move the wheelchair too fast and do not make sudden turns.
- Do not stand on the footrests when getting in or out of the wheelchair.
- Avoid lifting the wheelchair using its moving parts to prevent injury to the user, caregiver, or wheelchair damage
- Refrain using the stroller with the backrest folded. This alters the wheelchair's balance and could cause tipping. Only fold the backrest for storage or transportation

### COMMENTS:

The power wheelchair is not intended to be used in the following cases:

- (1) In restricted cases per manual: steep gradients, large obstacles, highways, or motor lanes.
- (2) On certain surfaces like wet grass on slopes where wheels may slip
- (3) When the control system or other important parts need to be repaired.

**NOTE:** Check for shipping damage. If found, DO NOT use the product. Contact the manufacturer for guidance.

**NOTE:** Improper use may cause tipping. Follow boarding/moving instructions.

### ATTENTION:

Regarding a product-related 'serious incident' leading to or potentially leading to:

- a) death of a patient, user or other person or
- b) temporary or lasting decline in the health of the patient, user, or others
- c) a serious threat to public health

Report this 'serious incident' to the manufacturer and the relevant authority in the user or patient's country. In Poland, contact the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

### ATTENTION:

If experiencing discomfort or unclear symptoms while using the medical device, seek advice from a healthcare professional

**ATTENTION:** it is forbidden to use the product in a way other than for its intended purpose!

**Application:** The product is designed for comfortable transportation of individuals with limited mobility. It's intended for single-person use

**Contraindications:** Physical or mental limitations (e.g. visual impairment) hindering safe wheelchair operation

### STORAGE

The product should be stored in a dry, cool room, away from moisture and direct sunlight.

### TARGET PATIENT GROUP

People with conditions or injuries the device is designed for (refer to intended use section). It can be bought by the user or recommended by a doctor/therapist. Consider available sizes, functions, indications, contraindications, and manufacturer information.

### 2. PRODUCT DESCRIPTION

The power wheelchair includes: a foldable frame for storage, an electronic control system with a panel, battery, two motors, drive mechanisms, and swivel wheels. There's a 24V 12AH battery with overload protection under the seat and a charging socket on the control panel's rear. The wheelchair has a brake lever for stopping and an anti-tip system

### 3. TECHNICAL PARAMETERS

Overall size (length x width x height): 1090mm x 640mm x 1350mm

Seat height: 540mm

Seat width: 430mm

Seat depth: 440mm

Armrest height: 200mm

Back height: 800mm

Battery weight: 2.5 kg

Weight of the wheelchair with the battery: 37 kg

Maximum speed: 6km/h

Braking distance at 6km/h:  $\leq 1500$ mm

Maximum turning radius:  $\leq 1200$ mm

Maximum load: 120kg

Range: 20km

Static stability:  $\geq 9^\circ$

Dynamic stability:  $\geq 6^\circ$

Permissible elevation:  $6^\circ$

Motors: 24V / 250W x 2

Battery: 24V/120Ah x 1

Control panel maximum output current: 50A

Battery output current: 2A

Front wheel: Tires outer diameter: 190mm

Rear wheel: Pneumatic tires outer diameter: 406mm

Overall width when folded: 390mm

### INFORMATION

This symbol indicates the user's maximum weight

### Troubleshooting

The electric wheelchair features a fault alarm system. When damage occurs, the LED on the control panel illuminates and an alarm sounds, aiding in fault detection as per Table 1. If subsequent faults arise after earlier rectification, seek advice from authorized customer service.

**Table 1** Basic Troubleshooting

Alarm	Number	LED Cause
1	1 LED	Low battery
2	2 LEDS	Left motor fault
3	3 LEDS	Left brake fault
4	4 LEDS	Right motor fault
5	5 LEDS	Right brake fault
6	6 LEDS	Control panel overload
7	7 LEDS	Joystick fault
8	8 LEDS	Control panel fault
9	9 LEDS	Control panel fault

**Table 2-** For proper maintenance, inspect the electric wheelchair before each use and at specified intervals per Table 2 for weekly, monthly, and semi-annual checks.

	Every use	Weekly	Monthly	Every six months
Each part			o	
Turning, driving, settings, assembly, etc.		o		
Brakes	o			
Cable connections		o		
Charging the battery	o			
Operation of the front wheels		o		
Pneumatic rear wheels		o		
Tire wear			o	
Tire damage	o			
Wear of handles, backrest and seat	o			
Engines				o
Control Panel		o		
Purity	o			

**4. ASSEMBLY**

Unpack the electric wheelchair and unfold it by extending the backrest outward. Raise the backrest using the handles at the back until it locks. Expand the wheelchair frame outwards and firmly press down on both sides of the seat. To secure the upper backrest, position the two tubes beside the seat base (refer to fig. 1, 2).

2. Put the cushions on the seat and backrest (see fig. 3).

3. Loosen the screws on the control panel bracket and place it at the armrest. Secure the control unit by tightening the screws (refer to fig. 4). The control unit can be mounted on either the right or left side.

Connect the control unit's cable plugs to the matching battery connections, then secure the cables to the frame using plastic ties (refer to Fig. 5 and 6).

5. Attach footrests to the frame's left and right tubes (refer to fig. 7). Rotate the footrests forward to secure them in the extended position and install the anti-tip wheels (see Figures 7 and 8).

6. Alter the footrest height by loosening the locknuts on both sides, removing the bolts (refer to fig. 9), adjusting the component supporting the footrest to the desired height, and then securing by reinserting the bolts and tightening the nuts.

IU\_AT52313

**5. OPERATION**

**Control panel**

The control panel is crucial for the power wheelchair, containing all essential electrical elements for operation (refer to Fig. 11). Positioned on an armrest, it links to both batteries and motors. Its components include:

1. Joystick
2. Battery indicator
3. On/off button
4. Speed indicator
5. Acceleration button
6. Mute button
7. Horn
8. SOS BUTTON

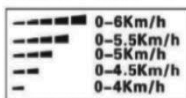
11.

**Joystick**

The joystick primarily controls wheelchair speed and direction. Moving it farther from the center increases speed. Releasing it centers the joystick and applies the brake automatically.

**Acceleration/deceleration buttons**

When activated, the speed indicator shows the maximum speed. The user can adjust this range by the number of lights shown. Pressing the acceleration (or deceleration) button increases (or decreases) the speed.

12.

**Horn**

After pressing the button, the horn will sound.

**Brake**

For braking, pull the brake lever to press against the wheels, stopping the wheelchair. When in motion, ensure the lever isn't pressed against the wheels by pushing it forward (refer to figure 13).

13.

**Seat belt**

For safety, fasten your seat belt when using the wheelchair. Push the belt in until it clicks (refer to figure 14).

**Position leverage**

Pull the lever to adjust the angle of inclination and return to the standard position (see figure 15).

14.

15.

### Footrests

The wheelchair has footrests with stepless adjustment and adaptable height (see Figures 16 and 17).

### 6. Precautions

Ensure the control panel and joystick are properly installed and centered. Once seated, fasten seat belts, lean against the backrest, and lift your head for safety. After turning on the wheelchair, verify the suitable speed on the control panel's indicator. Adjust using the acceleration or deceleration button if needed. Start with a slower speed for safety, gradually increasing. If the wheelchair moves unintentionally, release the joystick to halt the wheelchair (if the joystick is functional)

Warning: Keep your feet on the footrest only;

Warning: Do not extend your arms beyond the armrests while driving.

Warning: Before using the wheelchair, make sure it is locked with the brakes.

### 7. Folding/unfolding and storing the wheelchair

The wheelchair must be folded for transport and storage as follows:

1. place the wheelchair on the ground. Disconnect the power supply.
  2. lift the footrest and rotate outwards by 90° (see figure 18).
  3. tilt the footrest frame section outwards, rotate 90° and pull (see figure 19).
  4. press the backrest folding lock on both sides of the handle (see figure 20).
- Fold the backrest and frame of the pram by lifting upwards by the centre of the seat (see figure 21).

### 8. Charging the battery

The battery can be quickly and easily recharged using the separately supplied electric wheelchair charger.

**ATTENTION! Use only the provided charger to charge the wheelchair battery. Avoid using other chargers like car chargers.**

Charge the battery with the separately supplied charger:

- Make sure that the control panel is turned off and that the brake lever is applied.
- Connect the three-pole plug to the three-pole socket on the control panel (see figure 22).
- Connect the charger plug to the power socket.
- Charging is signaled by the LED turning red. When the battery is fully charged, the LED turns green.
- The recommended charging time is 8-12 hours.
- When charging is complete, disconnect the charger plug from the electrical outlet and then disconnect it from the plug on the control panel. Keep the charger, cables and plugs together in the pocket behind the backrest.

22.

23.

**9. Overload protection**

When the motors are overloaded, the overload protection cuts off the current to safeguard the motors and their electrical components. This setting requires specialized attention: unscrew the screws on the battery box, replace the backup fuse, cover the battery box, and tighten the screws.

**10. Seat belt**

To ensure safety when using a power wheelchair, seat belts must be fastened.

**11. Cleaning and maintenance after use**

- (1) Turn off the power (preferably disconnect all cable connections).
- (2) Keep the wheelchair away from children.
- (3) The wheelchair should be stored at normal temperature to prevent damage and prolong its service life.
- (4) Use a clean and soft cloth to clean the frame. The frame must be dried.

**Included in package:** electric wheelchair, 3 pillows (2 for backrest, 1 for seat), footrests with belt, controller, anti-tip 'mustache', instruction manual.

**HOW TO DISPOSE OF THE PRODUCT**

This medical device is an electrical product. Properly dispose of electrical and electronic equipment. Note: contains batteries.

**12. WARRANTY**

Item	Warranty Duration
Electrical parts (motor, controller, charger)	12 months from the date of purchase
Wheelchair frame	12 months from date of purchase
Battery	6 months from date of purchase

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WARRANTY CARD

Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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