AT52333

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Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before becoming familiar with and understanding this manual. If you do not understand the warnings, notes, and recommendations, please contact a healthcare professional or the seller to avoid product damage or personal injury.

WARNING: Check all parts for damage caused during transport. If such damage is found, DO NOT use the product. For more information, contact the seller.

TARGET GROUP OF PATIENTS

Individuals suffering from diseases, dysfunctions, or injuries for which this product is intended for treatment/rehabilitation or compensation (see the section on the purpose of the product in this user manual). The product can be purchased independently by the user or on the recommendation of a doctor, therapist, or other specialist. Whether the product is purchased independently or recommended by a doctor/therapist/other specialist, the available sizes/necessary features and product variants, as well as indications and contraindications for use, and information provided by the manufacturer, should be taken into account.

APPLICATION

The product is designed for the movement of disabled individuals with limited mobility or a complete inability to walk. The wheelchair is designed to transport one person.

INDICATIONS

Significant mobility limitations: Individuals with severe muscle strength limitations, for example, due to neurological diseases, muscular disorders, or pain syndromes, which prevent or significantly hinder independent movement.

Neurodegenerative diseases: Individuals suffering from conditions such as multiple sclerosis, Parkinson's disease, muscular dystrophies, or ALS, which lead to progressive loss of muscle control and strength.

Fatigue: Individuals for whom prolonged movement or manual operation of a wheelchair is too burdensome due to illness or health condition, causing rapid fatigue and exhaustion.

Age-related limitations: Elderly individuals with reduced mobility and strength due to age may benefit from an electric wheelchair to maintain independence and mobility.

Rehabilitation and recovery periods: Individuals undergoing rehabilitation after injuries or surgeries, who are temporarily unable to move independently, can use an electric wheelchair as part of their treatment and recovery process.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment): that prevent safe handling of the product.

Individuals who have difficulty judging distances or spaces may find it challenging to safely maneuver an electric wheelchair.

Electric wheelchairs require certain manual and coordination skills to operate the joystick or other control devices. Individuals unable to effectively operate these devices may not be able to use the electric wheelchair safely.

Although many people with cognitive impairments can safely use electric wheelchairs, severe disorders in some cases may make safe operation impossible.

Individuals unable to maintain a stable sitting position may require additional support systems to safely use an electric wheelchair.

Epilepsy or other conditions that may lead to sudden impairment of consciousness or reactions could be a contraindication.

If a person has coordination difficulties or cannot independently operate the wheelchair controls, it may be a contraindication.

Individuals with awareness disorders that could affect their ability to safely operate a wheelchair may also be unsuitable for its use.

In some cases, severe heart problems or blood pressure conditions may limit the safe use of an electric wheelchair, especially if emotions or stress could trigger undesirable reactions.

Conditions such as advanced multiple sclerosis or other diseases affecting muscle control may require specialized adaptive solutions in wheelchairs.

Problems with depth perception, spatial awareness, or general orientation may significantly limit the safety of using an electric wheelchair.

General poor health or recent surgeries, especially involving the spine or lower limbs, may require delaying wheelchair use until health improves.

LIST OF COMPONENTS

Electric wheelchair, remote control, user manual.

TECHNICAL DATA

Length: 86 cm

Width: 63 cm

Height: 96 cm (highest backrest adjustment point: 113 cm)

Folded length: 75 cm

Folded width: 63 cm

Folded height: 38 cm

Seat width: 43 cm

Seat thickness: 6 cm

Seat length: 42 cm

Backrest height: 41 cm

Armrest height: 24 cm

Weight: 29.5 kg

Front wheels: Solid, 8 inches

Rear wheels: Pneumatic, 12 inches

Motor: 350W x 2, brushed

Battery: 24V, 13A, lithium

Theoretical driving range: ≤20 km

Maximum speed: 6 km/h
Frame material: Aluminum
Seat material: Polyester fiber
Cushion: Foam + polyester cover
Front wheel: Aluminum alloy rim + solid PU
Rear wheel: Aluminum alloy rim + pneumatic rubber + plastic cap
Armrest: PU foam
Footrest: Aluminum alloy
Side bags: Oxford fabric
Side frame blue belt cover: Plastic
Maximum user weight: 120 kg

This symbol indicates the maximum user weight.

The provided dimensions may vary within ± 2 cm.

UNFOLDING THE WHEELCHAIR


Remove the wheelchair and spare parts from the packaging.

Install the controller on the driver's right armrest.

Hold the push tube with one hand and the seat frame with the other, pulling outward.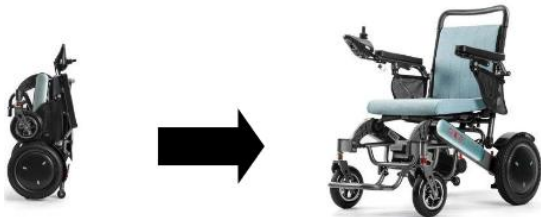

Lock the clamp (place the U-shaped clamp on the latch, then press the lever down).

Lock the armrest (press the armrest, then turn the small lever on the side of the armrest upward). (The button randomly selects two modes.)


FIRST OPERATION

Find an open space, such as a park, and ask an assistant for help during practice until you can operate the wheelchair proficiently. Ensure the power is off before getting in or out of the seat. Set the speed adjustment button according to your needs. It is recommended to set it to the lowest speed until you become familiar with the wheelchair's handling characteristics.

PRACTICE

Practice stopping, moving forward, and reversing. Move the joystick in the direction indicated in the diagram to move in the corresponding direction. First, practice moving forward at the lowest speed setting. After mastering forward movement, practice "S" turns. Next, practice reversing. Pay attention to speed control while reversing.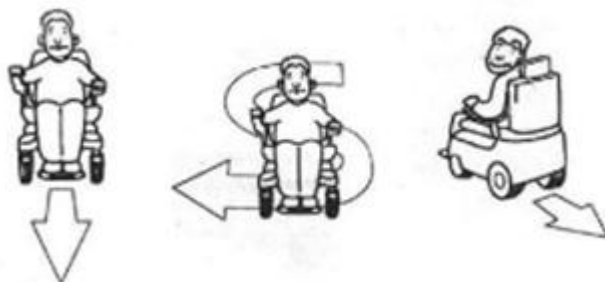

FOLLOW THESE INSTRUCTIONS TO MINIMIZE THE RISK OF LOSS OF BRAKE CONTROL OR SERIOUS DAMAGE CAUSED BY THE ELECTRIC WHEELCHAIR:

- Do not turn on communication devices such as CB radios and mobile phones while the wheelchair is on.
- Pay attention to transmitters such as radio or TV stations and avoid getting close to them.
- In case of unintended movement or brake release, immediately turn off the wheelchair for safety.
- Note that adding accessories or modifying the wheelchair may increase its susceptibility to radio wave interference.
- Report any cases of unintended movement or brake release to the seller and check for nearby sources of radio waves.

Immediately turn off the power to the electric wheelchair in the following situations:

- Sudden movement.
- Loss of directional control.
- Brake failure.

SAFETY INSTRUCTIONS

Do not transport passengers.

Do not use the wheelchair on slopes.
 Do not operate under the influence of alcohol.
 Do not tow other vehicles.
 Do not turn on or use handheld communication devices such as radios or mobile phones.
 Do not use the wheelchair without reading the user manual.
 Do not exceed the maximum safe incline angle of 13°.
 Do not operate the joystick irregularly on inclines.
 Do not transport passengers or exceed the maximum user weight while driving.
 Do not press the ON/OFF controller button while driving.
 Do not drive on soft terrain (loose soil, gravel, deep grass).
 Do not attempt to climb stairs higher than 50 mm unless a ramp is installed.
 Do not disassemble the wheelchair unless the electromagnetic brakes are engaged, and the controller is off.
 Do not use the wheelchair when no one is seated in it.
 Do not drive the wheelchair on the roadway, except on sidewalks.
 Do not change direction or reverse until the wheelchair has stopped.
 Always set the lowest speed when descending a slope.
 Always wear a seat belt.
 Always keep your feet on the footrest while driving.
 Do not stand on the footrest.
 Always ensure that the batteries are fully charged before setting out.
 Charge the wheelchair in a well-ventilated area to avoid risk.
 If the central wheel bolt is loose, replace it with a standard bolt using a wrench.
 When going down a slope, move the seat as far forward as possible.


WARNINGS AND TIPS

Do not use the electric wheelchair if the motor is unlocked.
 Do not disassemble the motor, as it may affect the vehicle's performance and cause damage.
Precautions when driving on slopes:
 Do not climb slopes exceeding 12°.
 Do not reverse when climbing a slope, as this may cause tipping.
 Do not drive on side slopes exceeding 3°, as this can easily lead to tipping.

JOYSTICK AND CONTROLLER OPERATION INSTRUCTIONS

Installation:

Attach the controller to the controller connection.
 Adjust the length of the controller connection tube, then tighten the nut.
 Connect the plug to the left and right motors and the battery.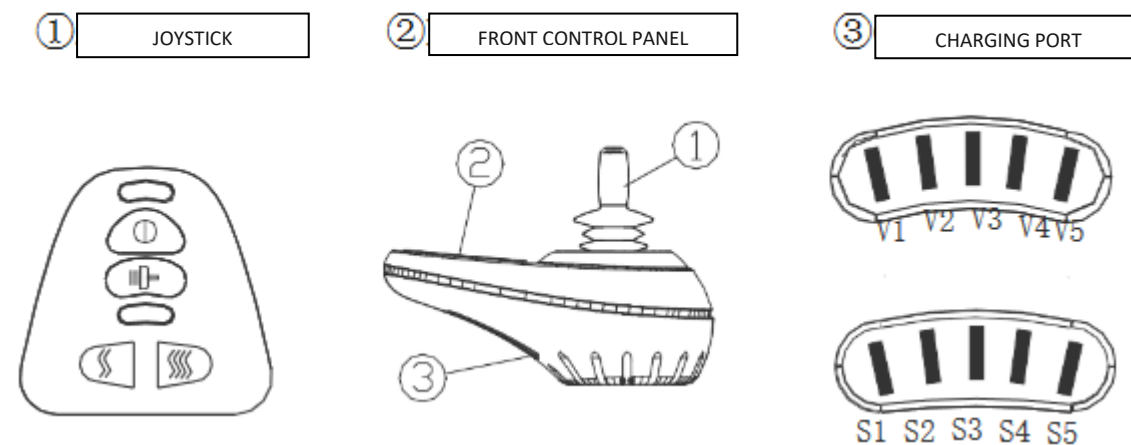

Controller Operation:

Avoid hitting the controller, especially the joystick. Do not strike the controller or joystick while driving.

Ensure the control system is secure and reliable while operating the wheelchair.

Do not store the control system or its components in a damp environment for extended periods.

If the control system comes into contact with food or drink, clean it immediately.

Cleaning:

Use diluted disinfectant and wipe with a cloth to clean the control system and joystick.

Be especially careful when cleaning the joystick.

Do not use abrasive or similar cleaning agents.

POWER ON

Press the ON/OFF button, and the battery charge indicators will light up from left to right.

**POWER OFF**


Press the ON/OFF button, and all LED lights will turn off.

**SPEED ADJUSTMENT**

Depending on the user's habits and environment, the speed of the electric wheelchair can be adjusted. The speed indicator shows the maximum speed, and adjustments can be made by pressing the speed decrease or increase buttons.

**HORN**

Pressing the horn button activates the sound, which will continue as long as the button is held down.

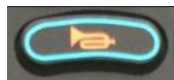**BATTERY LEVEL INDICATOR**

When the electric wheelchair is turned on, the battery level indicator operates, showing an approximate value of the remaining energy. As shown in the diagram, the battery is fully charged when all the LEDs are lit.

**REAR WHEEL CLUTCH FORK**

Clutch Mode: In electric mode, after stopping the wheelchair, the mechanical gear brake is used, making manual pushing difficult. To enable easy manual pushing, move the "red lever to manual mode." In this mode, the electric drive is disabled. Otherwise, the wheelchair will be difficult to maneuver.

Manual Mode: Simply move the red lever to manual mode.

NOTE: Try to keep the wheelchair in electric mode. The clutch function is intended only to enable manual pushing of the wheelchair when necessary (e.g., in shops or crowded places).

Pushing the Wheelchair:

Move the motor clutch forks to the left or right.

Push the wheelchair.

FOLDING THE WHEELCHAIR

Pull the small lever on the outer side of the armrests downward to loosen them. (Some models have buttons that can be pressed to release the armrest.)

Loosen the rear locks and lower the U-shaped lock.

Using one hand to hold the pushing frame and the other to grip the seat frame, fold the wheelchair completely with both hands.

ARMRESTS

Pull the small lever on the outer side of the armrests downward to set them in a loose position, allowing them to tilt backward. For users who cannot support their legs or operate the armrests independently, it is recommended that an assistant helps with their operation.

Important: Any assistant helping the wheelchair user should be properly trained in wheelchair operation.

BATTERY

Removing the Battery:

Unplug the power connection.

Hold the locking latch to detach the battery from the frame, then slide out the battery compartment.

Using the Battery:

The electric wheelchair uses a 24V sealed, maintenance-free lithium battery designed for deep discharges. While similar to car batteries, it cannot be replaced by a car battery as it is not designed for prolonged discharge and is not safe for use in electric wheelchairs.

BATTERY CHARGING AND DISCHARGING

Fully charge the battery before first use to achieve 90% efficiency.

When using the wheelchair at home or in the yard, start slowly and do not venture too far until you are familiar with the control system.

Fully discharge the battery.

Charge the battery for 8–14 hours, then use the wheelchair again. This process allows the battery to reach over 90% of its potential.

After 4–5 full charging and discharging cycles, the battery will achieve full efficiency and provide long-lasting performance.

Important Battery Information:

Regularly charging the battery ensures reliable operation and long life. Always keep the battery fully charged whenever possible.

Infrequent or incomplete charging can cause permanent battery damage, affecting reliability and reducing lifespan.

If you do not frequently use your wheelchair, charge the battery at least once a week.

CHARGER

The charger is a key component of the wheelchair, providing power safely, quickly, and easily.

Maintenance Recommendations:

Motor Carbon Brushes: Check the brushes every six months at a local service center.

Joystick/Controller: Protect the controller from moisture, which can cause damage.

Battery Wires: Ensure the wires are not corroded.

Cleaning: Clean the wheelchair with a damp cloth; do not use water.

REMOTE CONTROL

PAIRING THE REMOTE CONTROL

a) Press the power button on the wheelchair to turn it on.

b) Hold the power button on the remote control for 5 seconds (ensure two AAA batteries are installed).

c) Press the speed decrease button on the joystick until the middle light illuminates.

d) Press the horn button to automatically pair the devices.

e) Press the down arrow button until the power button on the remote control stops flashing.

f) When the joystick light turns off, pairing is complete.

g) You can now use the remote control instead of the joystick.

NOTE:

In the event of a "serious incident" related to the product, which directly or indirectly resulted, could have resulted, or may result in any of the following:

- a) Death of the patient, user, or another person, or
- b) Temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) A serious public health threat,

the "serious incident" must be reported to the manufacturer and the competent authority in the country of residence of the user or patient. In Poland, the competent authority is the **Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products**.

NOTE:

If pain, allergic reactions, or other troubling or unclear symptoms occur while using the medical device, consult a healthcare professional.

NOTE:

It is prohibited to use the product in a manner other than as intended.

ELECTRIC WHEELCHAIR FAILURES AND SELF-CHECK

In the event of an electric wheelchair failure during operation, turn off the power before performing any checks. A complete loss of power is indicated by all LEDs on the control panel turning off.

Steps for Checking:

Verify that the controller plug is correctly connected and not loose.

Check whether the controller plug connecting to the battery box is securely connected. Reconnect this plug (hold the connector, not the wire, to avoid wire damage).

If the power does not return after these checks or if there are any questions, contact the service center immediately.

Note:

The controller has a diagnostic system that monitors the controller and motor. If any component fails, the system will display an error on the controller.

CHECKLIST

Before each use, check the following components for damage and ensure they are in the correct position:

- Backrest
- Armrests
- Controller position
- Footrests
- Manual and power mode clutch
- Wheelchair battery charge level

Monthly check:

Inspect the following components to avoid wear and damage:

- Manual and power mode clutch
- Brakes
- Tires
- Controller and charger connections
- Presence of all screws

Semi-annual maintenance:

A full inspection is recommended every six months.

CLEANING AND MAINTENANCE

Maintenance tasks include:

Cleaning the wheelchair.

Inspecting the tires and battery.

Charging the battery.

For advanced maintenance, contact a service center. We recommend a full check of the electric wheelchair every six months and an annual inspection.

CLEANING THE ELECTRIC WHEELCHAIR

Clean the electric wheelchair regularly. Components frequently touched by the user (e.g., seat cushion, armrests, controller) should be wiped with a clean, slightly damp cloth. Do not use organic solvents for cleaning.

If the wheelchair is used by a patient, clean it weekly. For a wheelchair used by someone with an infectious disease, disinfect it with appropriate disinfectants.

For dirt on surfaces, wipe with clean water or mild detergent and dry with a cloth.

Do not use alcohol, gasoline, kerosene, or other corrosive or volatile chemical solvents for cleaning, as these may severely damage the wheelchair's appearance and internal structure.

Never use a high-pressure washing gun to avoid damaging internal electronic components and wiring.

TIRES

Regularly check tire pressure and wear. If the tread depth decreases to 1 mm, replace the tire.

BATTERY

Ensure the battery is frequently fully charged. To extend its life, avoid recharging it until it is completely discharged.

CHARGER

The charger is a vital component of the wheelchair, providing energy in a safe, fast, and simple way.

Maintenance Recommendations:

Motor Carbon Brushes: Inspect the brushes every six months at a local service center.

Joystick/Controller: Protect the controller from moisture, which can cause damage.

Battery Wires: Ensure the wires are not corroded.

Cleaning: Clean the wheelchair with a damp cloth; do not use water.

STORAGE

The product should be stored in a dry place, away from direct sunlight.

Do not store the electric wheelchair in humid conditions, such as near a bathroom.

If the wheelchair gets wet, wipe it dry immediately.

Keep the wheelchair away from strong magnetic fields and large electrical devices, such as radio or TV stations, underground radio stations, radio or telephone transmitters.

Avoid exposure to sources of electromagnetic interference, and keep a safe distance from them.

Protect the wheelchair from moisture and sunlight, and store it away from heat sources.

The wheelchair should be stored in a location free from rain, outdoor conditions, or humidity to avoid damage to electrical components caused by moisture.

Storage Conditions:

Ambient Temperature: -40°C to +55°C

Relative Humidity: ≤ 80%

Air Pressure: 86 kPa to 106 kPa

DISPOSAL

This product contains electronic components. At the end of its service life, it must not be disposed of in regular trash bins. For proper disposal, deliver the product to an appropriate collection point for electrical and electronic equipment.

Proper disposal prevents potential harm to the environment and human health and complies with legal regulations concerning the recycling of electrical and electronic equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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