# COMPRESSOR NEBULIZER BREEZE

# MANUAL

CE 0123

EC

REP

Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, Germany

Shenzhen Aeon Technology Co., Ltd.  
RM6H02, Block 27-29, Tianxia IC Industrial Park, Majialong,  
No. 133 of Yiyuan road, Nantou Street, Nanshan District,  
518052 Shenzhen, PEOPLE'S REPUBLIC OF CHINA

ECNOOI

# COMPRESSOR NEBULIZER

## WHAT IS COMPRESSOR NEBULIZER

Compressor Nebulizer is a compact medical device designed to efficiently deliver physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment. This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

Your Compressor Nebulizer should be used under the supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this manual to learn about the features of this product. Any use of this product other than its intended use should always be avoided.

## IMPORTANT SAFETY INSTRUCTIONS

### READ CAREFULLY AND STORE THESE INSTRUCTIONS FOR FUTURE USE.

- Before connecting the appliance to a power socket, check that the technical details on the appliance label match the electrical voltage in the socket.
- Only plug the appliance into a correctly grounded socket.
- This appliance is designed for use indoors. Do not use this appliance for commercial purposes. Do not use the appliance in industrial surroundings or outdoors!
- Do not use the appliance for any other than its intended purpose.
- Always turn off the appliance and disconnect it from the power socket if you leave it without supervision or if you will not be using it and before cleaning.
- To avoid any injury by electrical shock, do not clean the appliance under running water and do not submerge in water or any liquid.
- Ensure that the power cord does not come into contact with water or moisture or heated surface.
- Do not disconnect the appliance from the power socket by pulling on the power cord. This could damage the power cord or the power socket. Disconnect the cord from the power socket by gently pulling out the plug of the power cord.
- If the power cord is damaged, have it replaced at a technical service center. It is forbidden to use the appliance if it has a damaged power cord or plug.
- To avoid the danger of injury by electrical shock, do not repair the appliance yourself or make any adjustments to it. Have all repairs or adjustments of the appliance done at an authorized service center. Tampering with the appliance during the warranty period may void the warranty policy.
- Keep the air vents open. Do not place the unit on a soft surface where the openings can be blocked.
- If the medicine cup is empty, do not attempt to operate the unit.


## CHILDREN AND FRAIL INDIVIDUALS

- In order to ensure your children's safety, please keep all packaging (plastic bags, boxes, polystyrene etc.) out of their reach. **WARNING!** Do not allow small children to play with the foil as there is a danger of suffocation!
- This device is not intended to be used by individuals (including children) who have restricted physical, sensory or mental abilities and/or insufficient knowledge and/or experience, unless they are supervised by an individual who is responsible for their safety or have received instructions on how to use the device.
- Children should be supervised at all times in order to ensure that they do not play with the device.

## Compressor

- 1) Jet
- 2) Medicine cup
- 3) Air tube
- 4) Angled mouthpiece
- 5) Power switch
- 6) Tube connector
- 7) Crescent-shaped groove
- 8) Air filter & cover
- 9) Air vents
- 10) Power cord

## Nebulizer

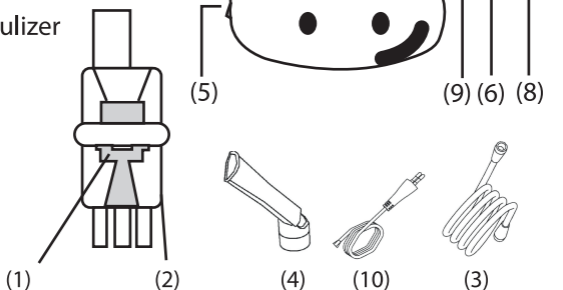### OPERATING YOUR COMPRESSOR NEBULIZER

Note: Prior to initial operation, the nebulizer should be thoroughly cleaned.

- Place your Compressor Nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated.
- Gently twist the top part of the nebulizer counter-clockwise to disassemble the nebulizer.
- Fill the bottom section of the nebulizer with the medication prescribed by your physician. Be sure that the cone is put inside the bottom section.
- Gently twist the top part clockwise to reassemble the nebulizer. Be sure the two sections fit well.
- Attach one end of air tube 3) to the base of the nebulizer.
- Insert the air tube 3) to the crescent-shaped groove 7). Make sure that the nebulizer is stably stand upright over the compressor.
- Attach the other end of air tube 3) to the tube connector located on the front of the compressor.
- Attach mouthpiece 4) or mask per your choice to the top section of nebulizer.
- Plug the power cord into an appropriate electrical outlet. Make sure at this stage, the power switch is at "Off" status.
- Press power switch to begin your prescribed treatment.

#### Important:

The compressor motor has a thermal protector, which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- + Switch off the unit.

- + Unplug the unit from the electrical outlet.
- + Wait 30 minutes for the motor to cool down before another treatment. Make sure the air openings are not obstructed. When treatment is finished, shut off the unit and unplug it from the electrical outlet.

## **CLEANING**

It is recommended that the nebulizer, mouthpiece and mask be thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

### **Rinsing (after each treatment):**

1. Disconnect the air tube 3) , nebulizer, mouthpiece 4) and mask.
2. Gently twist the nebulizer to open it.
3. Rinse the nebulizer, mouthpiece 4) and mask with water.
4. Dry them with clean soft towel or let it air dry.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

### **Disinfections:**

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

- Use one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece, and mask.
- Complete Rinsing Steps 1-3.
- Wash nebulizer, mouthpiece 4) and mask in warm water and a mild detergent. Then wash them in hot tap water.
- Submerge these parts in the vinegar and water solution for thirty minutes.
- Complete Rinsing Steps 3-5.

### **Cleaning the compressor:**

- Wipe daily with a damp cloth.
- Do not use any powdered cleaners or soap pads, which may damage the finish.

### **Filter change:**

- Do not use cotton or any other materials. Do not wash or clean the filter. Only use filters supplied by your distributor. And do not operate without a filter.
- Change the filter every 30 days or when the filter turns gray.

## **STORAGE**

- Do not store the unit under direct sunlight, high temperature or humidity.
- Keep the unit out of reach of small children.
- Always keep the unit unplugged while not in use.

## **SPECIFICATIONS**

Power: 220-240V, 50 Hz

Current: 0.8A

Medication Capacity: 10 ml

Particle Size: 0.5 to 10 um

Sound Level  $\leq$  65 dBA

Average Nebulization Rate  $\leq$  0.3 ml/min

Compressor Pressure Range: 29 to 52 Psi (200 to 360 KPa / 2 to 3.6 bar)

Operating Pressure Range: 11.6 to 19 Psi (80 to 130KPa / 0.8 to 1.3 bar)

Liter Flow Range  $\leq$  8L/Min

Operating Temperature Range: 10°C to 40°C (50°F to 104°F)

Operating Humidity Range: 10 to 93% RH

Storage Temperature Range: -25°C to 70°C (-13°F to 158°F)

Standard Accessories: Nebulizer, Air Tube, Adult Mask, Child mask, Mouthpiece.

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## WARRANTY

1. This warranty is issued by EDC Poterek Sp. Jawna with address: ul. Poznańska 129/133, 05-850 Ożarów Mazowiecki, Phone: 022-7213571 w.10 – called hereinafter Esperanza.
2. This warranty confers upon you specific rights and remedies against Esperanza. Please read this document carefully, by making a claim under this warranty, you will be deemed to have understood and accept its terms.
3. Esperanza warrants that any Esperanza product together with attached accessories is free from defects in materials and workmanship under conditions of its normal use.
4. If you make a claim under this warranty, Esperanza will, at its option, repair or replace any defective product into new one by using new or use factory-refurbished parts. In case of using factory-refurbished parts they will be equal to new products.
5. Defective product should be returned together with receipt, commercial invoice or any other proof of purchase to the place of purchase. It should include all accessories attached in original packing. Esperanza has right to reject warranty if any accessories will be missed.
6. Esperanza is committed to respond to the complaint within 14 calendar days of its notification in the point of sale. Esperanza will repair the defective product in the shortest possible time. If will be need to import from abroad of any parts which are necessary to repair this time may be extended.
7. In case of replacing defective product onto new one, Esperanza has right to replace it with newer model with the same technical parameters if the old one model was replaced by new one during warranty period.
8. If you make claim under this warranty, you are deemed to have agreed that repair and (upon availability) replacement, as applicable, under the warranty services described herein, is your sole and exclusive remedy vis-a-vis Esperanza.

## EXCLUSIONS:

### This warranty does not extend to:

1. Any product not manufactured by Esperanza
2. Any product that has been damaged or rendered defective as a result of:
  - use it in other way than normal intended use
  - modification or internal interferes or changes of product
  - service or repair by anyone other than Esperanza service
  - improper store, transportation or packing
  - improper installation of product or software
  - mechanical, physical, chemical etc. damages
3. In no event shall Esperanza be liable for any damages, loss of business, profits, data or loss based on use Esperanza products with other devices.
4. Products with removed warranty and other labels and products spent or have hard use marks.

All products are covered by Esperanza's 2-year manufacturer's warranty while product is purchased by the consumer (the consumer is a person who buys product for the purpose of not professional or commercial activity). In the case of purchase for the company (for purposes relating to the activities of professional or commercial) warranty is 1 year from date of purchase. Esperanza products have all the necessary safety certificates required in the EU.

According to Directive 2012/19/EU, disposal of waste electrical and electronic equipment together with household waste is prohibited. Their components need to be recycled or disposed separately. Otherwise, they can cause the release of hazardous and noxious substances which pose a threat to the environment and human health. The law requires each consumer to return unnecessary or waste electrical and electronic equipment to the producers, sales agents or waste treatment facilities. Detailed conditions are governed by national legislation. The above symbol placed on the product, in the user manual or on the packaging refers to this requirement. Thanks to proper segregation and compliance with the provisions relating to waste disposal each user contributes significantly to environmental protection.

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We, the manufacturer:

**Shenzhen Aeon Technology Co.,Ltd.**

RM6H02, Block 27-29, Tianxia IC Industrial Park, Majialong,  
No. 133 of Yiyuan road, Nantou Street, Nanshan District,  
518052 Shenzhen, China

Herewith, Declares that the product:

Product description: COMPRESSOR NEBULIZER

Model Number: A500LW07

Is technically identical with the following product imported from our company by EDC Poterek Sp. Jawna company:

Product description: COMPRESSOR NEBULIZER

Model Number: ECN001

Trademark: ESPERANZA

Manufacturers/Authorized representative

## DECLARATION OF CONFORMITY

We, the manufacturer:

**Shenzhen Aeon Technology Co.,Ltd.**

RM6H02, Block 27-29, Tianxia IC Industrial Park, Majialong,  
No. 133 of Yiyuan road, Nantou Street, Nanshan District,  
518052 Shenzhen, China

**We confirm that this product has CE.**

Herewith, Declares that the product:

**Product description: COMPRESSOR NEBULIZER**

**Model Number: A500LW07 (ECN001)**

The products is compatible with the following norms/standards:

**Directive 93/42/EEC on Medical Devices (MDD), Annex II excluding (4)  
(Devices in Class IIa, IIb or III)**

**Issued by TUV SUD Product service GmbH, Notified body: 0123,**

**EC Certificate No: G1 073767 0013 Rev. 00**

**This certificate is valid from 3<sup>rd</sup>, Feb, 2019 until 2<sup>nd</sup>, Feb, 2024**

Manufacturers/Authorized representative

2022-05 SHENZHEN