AT51019

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EN

READ ALL INSTRUCTIONS CAREFULLY BEFORE USE

The height-adjustable toilet chair is a modern product designed with people in mind with limited mobility, the assembly of which is quick, easy and does not require tools. The product is made of a stable steel frame, equipped with soft and comfortable armrests. The legs of the chair are finished with non-slip feet for stability. Included there is a bucket with a lid and a toilet seat with a lid.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

PRODUCT CHARACTERISTICS:

- frame made of high-quality steel,
- soft, comfortable armrests
- possibility of height adjustment
- non-slip feet
- a bucket with a lid
- easy to clean toilet seat

APPLICATION

The toilet chair allows patients who are confined to bed to satisfy their physiological needs. stay in bed. It can be used in close proximity to the bed or the area where the patient with limited mobility. In addition, the chair can be used without bucket over the toilet seat in the toilet.

COMMENTS AND WARNINGS

DO NOT USE IF YOUR WEIGHT EXCEEDS 136 KG.

KEEP AWAY FROM DIRECT SUNLIGHT AND HEAT SOURCES.

THE TOILET CHAIR IS DESIGNED FOR USE ON FLAT AND LEVEL SURFACES.

IT IS NOT PERMITTED TO STAND ON THE TOILET CHAIR.

CHECK THE EQUIPMENT REGULARLY FOR DAMAGE AND WEAR. IN CASE OF ANY DAMAGE, STOP USING THE PRODUCT IMMEDIATELY.

REPAIR AND DISINFECTION SHOULD ONLY BE CARRIED OUT BY PERSONS AUTHORISED PERSONNEL!

ALWAYS EMPTY THE CONTENTS OF THE STORAGE BOX IMMEDIATELY AFTER USE.

MAKE SURE THE FEET ARE IN GOOD CONDITION (DAMAGED OR MISSING FEET POSE A DANGER OF "SLIPPING").

BE AWARE OF THE PROPER CONDITION OF THE FEET (DAMAGED OR MISSING FEET POSE A DANGER OF "SLIPPING" WHEN SITTING DOWN OR STANDING UP FROM THE CHAIR).

ATTENTION:

In the event of a product-related "serious incident" which has directly or indirectly led, could have led or may lead to any of the following listed below:

- a) death of a patient, user or other person, or
- b) a temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

this 'serious incident' should be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult with a healthcare professional.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

After the device is decommissioned, the medical device can be disposed of as normal municipal waste, with the exception of electrical products - proceed in a manner appropriate for the disposal of equipment.


USAGE

1. Remove the product from the packaging. The product is delivered folded. (Pic. 1.1)
2. Place the bucket in the space provided for it, between the metal barriers.
3. The chair comes with two separate components that must be placed in the correct order on the railings. The board with the hole is placed first, followed by the upholstered cover. Before use the chair, the cover must be removed.
4. To change the height of the chair, adjust the height of each leg separately as follows: (Pic. 1.2).
 - a. Remove the latch 1.
 - b. Adjust the feet to the required position 2.
 - c. Lock the latch 1.

Before use, make sure that the plastic fixing clips are properly attached and that the legs of the chair are of the same height.

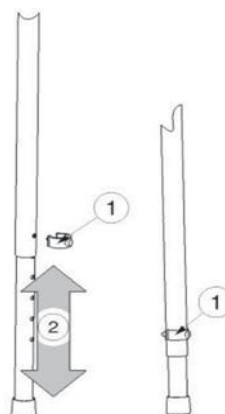Pic 1.1

Pic. 1.2

PRODUCT MAINTENANCE

1. Empty and disinfect the container and toilet seat after each use to avoid contamination.
2. Clean the chair frame with water and cleaning agents (avoid corrosive substances).
3. Clean the armrests with mild detergents.
4. The unused product should be stored in a dry place.

TECHNICAL PARAMETERS

Type: solid

Material: steel, plastic

Maximum distance from the floor to the armrest: 77 cm

Minimum distance from the floor to the armrest: 62 cm

Width between the legs: 50 cm

Maximum seat height: 57 cm

Minimum seat height: 42 cm

Seat dimensions: width: 42 cm, depth: 42 cm

Backrest measured from the seat: 40 cm

Backrest surface: 25 cm

Overall width: 65 cm

Overall height: 99 cm (highest setting), 85 cm (lowest setting)

Weight: 7.2kg

Maximum user weight: 136 kg

THIS MARK INDICATES THE MAXIMUM USER WEIGHT.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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